

Enjoy a fulfilled love life once again!

Enjoy your love life once again with plenty of fun and without the glitches. LANCY *Men* therapy has been proven by studies and, when used correctly, is

- easy to use
- without side effects
- and cost-effective.
- It is carried out discretely by the patient in their own home
- without the pressure of appointments
- and without the assistance of others.

Studies demonstrate the therapy's success: 58% improvement after 3 months of treatment

"The transcutaneous electrical stimulation of the striated pelvic floor muscles leads to erections capable of coitus after 3 months of treatment in 58% of patients (20 minutes/day)." (Derouet 1998)

93% improvement after 6 months of treatment

"93% (119/128) of the group that received the electrical therapy answered the question "Did therapy improve your erection?" with "yes", compared to the group of men that received Viagra, where only 20% (37/124) agreed."

(Prof. Dr. Sommer 2014)

"It is postulated that a special electrical therapy of the flat muscle cells in the corpus cavernosum and the striated pelvic floor muscles in selected men with erectile dysfunction can lead to the curing of the erectile dysfunction." (Prof. Dr. Sommer 2014)

You can order the therapy device from us, including accessories and a 12-month guarantee on the device itself, for €490.00. Costs for sale abroad are not included and must be clarified in advance.

Regain your potency.

Be a winner in every day life.



Elektromedizin

LANCY Elektromedizin
An der Bachmühle 4
74821 Mosbach

Phone: +49 6261 938693
Fax: +49 6261 915759

info@lancy-elektromedizin.de
www.lancy-elektromedizin.de



Successor certifications:
German Medicinal Devices Act (MPG), DIN EN 13485
Pre-qualification method



Pole- Position LANCY Men



For your potency

Pole-Position

LANCY *Men* 



More frequent. Longer. Love.

Noticeable improvement in your potency.

IT CAN HAPPEN...


or am I already impotent? The causes of erectile dysfunction vary, and must be examined by a doctor in order to find the appropriate treatment:

- Diabetes
- Onset of old age, calcification of the supplying blood vessels
- Hormonal and physical disturbances
- Leaks in the spongy bodies
- Restructuring of the binding tissue of the spongy bodies, e.g. after a long erection
- Damage caused by operations on the prostate
- Radiation and injury
- Alcohol abuse, nicotine addiction, side effects of medication
- Metabolic illnesses with neuropathy
- Some paraplegias
- Personal and work-related stress

WHAT CAN I DO? TREATMENT WITH LANCY *Men*

Which erectile dysfunctions does it help?

LANCY *Men* is suitable for the treatment of erectile dysfunctions if muscle cells in the spongy body system and the pelvic floor have been damaged or are weakened (organic nature). Important: even when spongy body auto-injection therapy

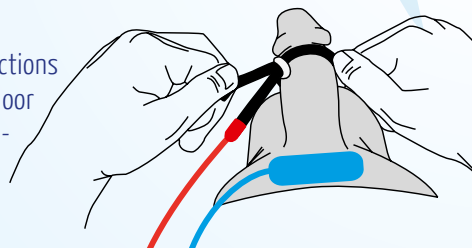

(SKAT) has failed, LANCYMen therapy can lead to renewed erections. SKAT treatment can become effective again at any time in connection with this treatment. It can be used as a prophylaxis (in order to prevent hardening of the tissue before or after prostate operations) or to strengthen and improve the quality of the erections.


How often and how long?

The individual treatments should be performed daily for 30 minutes for at least three months.

How does LANCY *Men* work?

The LANCY *Men* device works with a bi-phasal current. Thanks to a special switching device, it is particularly safe and gentle on the skin. The device is equipped with three different programmes, for which the frequencies are fixed. Depending on the diagnosis of the patient, the

The gentle LANCY *Men* stimulation current therapy works using low frequency, completely harmless currents. These are applied to the skin using adhesive electrodes, and there they stimulate nerves or muscle cells, such as for example in the figure to the left, for the application of programme 1.

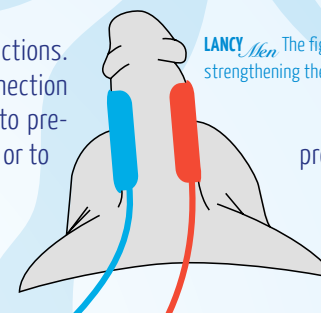

LANCY *Men*. The figure to the left shows programme 2: strengthening the muscle cells of the spongy body.

program selection and the application time can be adapted individually. The strength of the current can be adjusted and is based on the electrode system. It is adjusted in such a way that the stimulation is noticeable yet pleasant, and is not painful.

When is it not suitable?

I may not use LANCY *Men* or may only use it after consultation with a doctor in the following cases – contraindications for users with:

electronic implants such as pacemakers or pumps, cardiac arrhythmias, epilepsy, skin diseases in the area of application of the electrodes, malignant diseases in the area of application.

The figure to the right shows programme 3: strengthening the muscle cells of the pelvic floor.

