

Consistence against incontinence

Consultation | Prevention | Therapy | Pelvic floor exercises | Presentations

Investment in more quality of life

Together with doctors in all fields and university clinics such as Heidelberg, Mannheim, Stuttgart and Tübingen, we are always working to improve treatment methods in the event of incontinence. For patients, TIBIALIS THERAPY offers an opportunity to improve their incontinence problems – even after effective medicinal treatment or a stay in a clinic.

When am I NOT ABLE TO USE TIBIALIS THERAPY? –
Contraindications: Pacemaker, pregnancy, metal implants in the immediate vicinity of the adhesive electrodes.

You can order the therapy device from us, including accessories and a 12-month guarantee on the device itself, for €498.00. Costs for sale abroad are not included and must be clarified in advance.

A life full of enjoyment – without limitation!

Consistence against incontinence

Consultation | Prevention | Therapy | Pelvic floor exercises | Presentations



Elektromedizin

LANCY Elektromedizin
An der Bachmühle 4
74821 Mosbach

Phone: +49 6261 938693
Fax: +49 6261 915759

info@lancy-elektromedizin.de
www.lancy-elektromedizin.de



Successful certifications:
German Medicinal Devices Act (MPG), DIN EN 13485
Pre-qualification method



Consistence against incontinence

Consultation | Prevention | Therapy | Pelvic floor exercises | Presentations

Urinary and anal incontinence – the new therapy

With LANCY TIBIALIS THERAPY to achieve better quality of life & more freedom

Responsible for the content: Brigitt Lantitzberg, photos: Theresa Rundel, fotoia.de



Elektromedizin

*Birgitt Lantzberg,
Director of LANCY Elektromedizin,
at a training session for a
patient in tibialis therapy*



WE WILL HELP YOU

If the urge to urinate and expel stool has taken the joy out of life for you, robbed you of sleep and limited your freedom of movement, you are in the best of hands with us.

We will help you with new and easy-to-do TIBIALIS THERAPY. By stimulating the tibial nerve (Nervus Tibialis), the pelvic floor musculature that is responsible for urine and stool control is activated. This treatment method is significantly more comfortable than previous existing therapies involving vaginal and anal wands.

EASY APPLICATION, NOTICABLE EFFECT*

Using adhesive electrodes, which are attached to the ankle near the tibial nerve, the nerve is stimulated for approx. 20 minutes with short pulses of current, which are transferred to the pelvic floor. The musculature there is tensioned, which strengthens it, and in this way it has a positive influence on urine and stool control.

The frequencies with which the nerve fibres are treated depend on the type of incontinence. The doctor determines these individually in a case history and consultation appointment. Attaching the plate electrodes and operating the stimulator, a small device that generates the current necessary, can be done by the patient in their own at home after a short instructional session.

Treatment occurs over a period of three months, once each day.

IN GOOD HANDS

For many years, our concern and our expertise has been used to give those suffering from incontinence a bit of their quality of life and freedom back. We don't leave anything to chance, and we analyse which maladies are involved and which therapies should be considered. Only after a careful examination of the case history do we decide whether therapy with plate electrodes or another alternative for pelvic floor stimulation can achieve success.